




SurGuard2™

Product Information Card

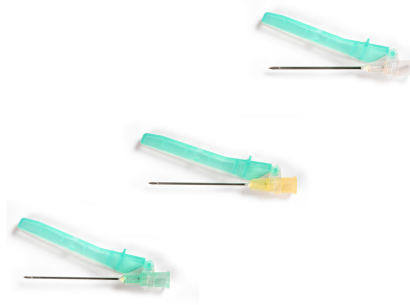


Description:

The SurGuard2™ needles are single use, latex-free, e-beam sterilized hypodermic needles, made from plastic materials and stainless steel.

The SurGuard2™ needles, when fitted with a standard luer slip or luer lock syringe are intended for the aspiration and injection of fluids for medical purposes. After withdrawing the needle from the body, the attached needle safety shield can be manually activated to cover the needle immediately after use to minimize the risk of an accidental needle stick.

Components:

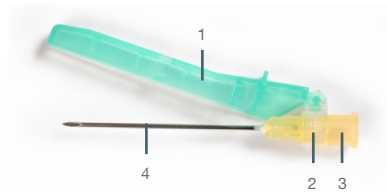

Description	Raw Materials
1. Sheath	Polypropylene (PP) random copolymer - with colorant
2. Collar	Polypropylene (PP) random copolymer
3. Hub	Polypropylene (PP) random copolymer - with colorant
4. Cannula	Stainless steel
Protector	Polypropylene (PP) random copolymer



Product Range:

Product Code	Colour	Cannula Ø		Cannula Length	
SG2+1825	Pink	18G	1.20 mm	1 inch	25 mm
SG2+1838	Pink	18G	1.20 mm	1 1/2 inch	38 mm
SG2+1925	Beige	19G	1.10 mm	1 inch	25 mm
SG2+1938	Beige	19G	1.10 mm	1 1/2 inch	38 mm
SG2+2025	Yellow	20G	0.90 mm	1 inch	25 mm
SG2+2038	Yellow	20G	0.90 mm	1 1/2 inch	38 mm
SG2+2125	Green	21G	0.80 mm	1 inch	25 mm
SG2+2138	Green	21G	0.80 mm	1 1/2 inch	38 mm
SG2+2225	Black	22G	0.70 mm	1 inch	25 mm
SG2+2238	Black	22G	0.70 mm	1 1/2 inch	38 mm
SG2+2325	Blue	23G	0.65 mm	1 inch	25 mm
SG2+2338	Blue	23G	0.65 mm	1 1/2 inch	38 mm
SG2+2516	Orange	25G	0.50 mm	5/8 inch	16 mm
SG2+2525	Orange	25G	0.50 mm	1 inch	25 mm
SG2+2538	Orange	25G	0.50 mm	1 1/2 inch	38 mm
SG2+2613	Dark brown	26G	0.45 mm	1/2 inch	13 mm
SG2+2713	Grey	27G	0.40 mm	1/2 inch	13 mm
SG2+3013	Yellow	30G	0.30 mm	1/2 inch	13 mm



Packaging:

Single unit:	blistered paper (film/paper)
Unit box:	100 pieces
	<i>Dimensions = 225 x 102 x 93 mm</i>
Shipping carton:	800 pieces
	<i>Dimensions = 394 x 216 x 214 mm</i>

Sterilization: With e-beam to a Sterility Assurance Level of at least 10⁻⁶

Storage: Shelf life of 5 years
** The batch number and expiry date are printed on the paper of the blister pack*
** Do not store at extreme temperature and humidity*

Precautions for Use: Check integrity of the unit package before use
 For single use only
 Do not attempt to re-insert a partially or completely withdrawn needle
 Dispose of safely after single use to avoid risk of infection

Status of Regulation: Class IIa, sterile product following MDD 93/42/EEC

Notified body: TÜV Rheinland 0197
 Manufacturer: TERUMO PHIIPPINES CORPORATION
 EC representative: TERUMO EUROPE N.V.
 Interleuvenlaan 40
 B - 3001 Leuven
 Belgium



Terumo Corporation
 44-1, 2-chome, Hatagaya
 Shibuya-ku, Tokyo 151-0072,
 Japan
 Tel.: +81 3 3374 8111
 Fax: +81 3 3374 8196

European Headquarters

Terumo Europe N.V.
 Interleuvenlaan 40
 3001 Leuven, Belgium
 Tel.: +32 16 38 12 11
 Fax: +32 16 40 02 49

European Offices

**Terumo Europe N.V.
 Benelux Sales Division**
 Interleuvenlaan 40
 3001 Leuven, Belgium
 Tel.: +32 16 39 25 80
 Fax: +32 16 39 25 99
 The Netherlands:
 Tel.: 0800 0220396
 Fax: 0800 0220414

Laboratoires Terumo France S.A.
 Parc Ariane - Bât. Uranus
 1, rue Hélène Boucher
 78284 Guyancourt Cedex, France
 Tel.: +33 130 96 13 00
 Fax: +33 130 43 60 85

Terumo Deutschland GmbH
 Ludwig-Erhard-Strasse 6
 65760 Eschborn, Germany
 Tel.: +49 6196 80 230
 Fax: +49 6196 80 23 200

Terumo Deutschland GmbH
 Zweigniederlassung Spreitenbach
 Bodenackerstrasse 3
 8957 Spreitenbach, Switzerland
 Tel.: +41 56 419 10 10
 Fax: +41 56 419 10 11

Terumo Europe España SL
 Avda. Juan Carlos I, Nº 13 - 7ª Planta
 Edificio Torre La Garena
 28806 Alcalá de Henares (Madrid),
 Spain
 Tel.: +34 9021 01 298
 Fax: +34 9021 01 358

Terumo Italia SRL
 Via Simone Martini, 143/145
 00142 Roma RM,
 Italy
 Tel.: +39 0651 96 14 20
 Fax: +39 0650 30 407

Terumo Sweden AB
 Nya Varvet, Byggnad 90
 Sven Källfelts Gata 18
 426 71 Västra Frölunda,
 Sweden
 Tel.: +46 3174 85 880
 Fax: +46 3174 85 890

**Terumo Denmark
 Filial of Terumo Sweden AB**
 Tel.: +45 7020 93 80
 Fax: +45 7020 94 80

Terumo UK Ltd
 Tamesis
 The Causeway
 Egham, Surrey
 TW20 9AW, United Kingdom
 Tel.: +44 1784 476 220
 Fax: +44 1784 476 201

**Terumo Europe N.V.
 Moscow Representative Office**
 BC Northern Tower
 13Fr., 10 Testovskaya Street
 Moscow 123317, Russia
 Tel.: + 7 495 988 4740
 Fax: + 7 495 988 4739

**Addresses subject to change,
 please consult our Website.**

HO 62GB-0807TE-III(09.11)E

© Registered Trademark.
 Published by Terumo Europe N.V.