

# stericlens®

0.9% w/v Sodium Chloride Ph. Eur.



Sprays  
any way  
up!



## sterile saline with no waste.

Stericlens is a versatile topical irrigation and wound cleansing system. Its narrow spray is designed to help remove debris and bacterial matter without having to touch the wound.

Stericlens features an innovative 'bag-in-can' technology which means there are no cold propellants, it can be sprayed in any position (even upside-down) and there's absolutely no wastage!

Available in 240ml and handy 100ml cans for maximum convenience.



**PRODUCT NAME:****Irripod®****CLASS:**

Wound irrigation fluid.

**SIZE:**

20ml pods.

**PRESCRIBING:**

Available on Drug Tariff.

**DESCRIPTION:**

A polythene ampoule (Pod) containing 0.9% w/v Sodium Chloride Ph. Eur. which is an odourless, clear, sterile aqueous solution.

**INDICATIONS:**

For the topical irrigation and cleansing of wounds.

**CONTRAINDICATIONS:**

Do not mix with any other fluid except as directed.

**WARNINGS AND PRECAUTIONS:**

- For external use only. Do not use for injections.
- Avoid direct contact between the nozzle and the surface of the wound.
- Keep out of reach of children.
- Do not use if the product is not clear.
- Do not use if the product has leaked before opening.
- For use on one occasion only - discard any leftover product.
- Not to be used with other irrigation fluids containing silver, lead or mercury.

**DIRECTIONS FOR USE:**

- Snap open the container (Pod) by twisting off the seal to reveal the nozzle.
- Hold firmly in a downward direction and squeeze to apply solution directly to the area to be irrigated, or invert and squeeze gently to deliver the product dropwise.
- Do not dilute.

**REGULATORY STATUS:**

Class IIb medical device with CE mark 0459.

**STORAGE AND DISPOSAL PRECAUTIONS:**

- The product should be kept between 5 - 25°C in a clean area.
- Keep well out of reach of children.
- Observe local disposal regulations and requirements.

**PRODUCT FROM:**

CD Medical Ltd

Email: [info@cdmedical.co.uk](mailto:info@cdmedical.co.uk)**ORDERING:**

Size	PIP Code	NPC Code
20ml x 25	313-7155	MRB 742

**PRODUCT NAME:****Stericlens®****CLASS:**

Wound irrigation fluid.

**SIZES:**

100ml and 240ml.

**PRESCRIBING:**

Available on Drug Tariff.

**DESCRIPTION:**

An aerosol containing 0.9% w/v Sodium Chloride Ph. Eur. which is an odourless, clear, sterile aqueous solution. Nitrogen is used as the propellant.

**INDICATIONS:**

For the topical irrigation and cleansing of wounds.

**CONTRAINDICATIONS:**

Except as directed, do not mix with any other fluid.

**WARNINGS AND PRECAUTIONS:**

- Do not use for injections.
- For external use only.
- Avoid direct contact between the nozzle and the surface of the wound.
- Keep out of the reach of children.
- Do not spray near or into the eyes.

**DIRECTIONS FOR USE:**

- Spray a small quantity through the nozzle before use.
- Spray area to be treated from a distance of about 10 to 12cm, working across the wound in order to thoroughly irrigate and cleanse it.
- Stericlens can be used through 360° and can even be held upside down if required.
- Replace cap when not in use.

**REGULATORY STATUS:**

Class IIa medical device with CE mark.

**STORAGE AND DISPOSAL PRECAUTIONS:**

Pressurised container:

- Do not expose to temperatures above 50°C.
- Protect from sunlight.
- Do not puncture or burn even when empty.
- Observe local disposal regulations and requirements.
- Keep out of the reach of children.

**PRODUCT FROM:**

CD Medical Ltd

Email: [info@cdmedical.co.uk](mailto:info@cdmedical.co.uk)**ORDERING:**

Size	PIP Code	NPC Code
100ml	305-9458	ELZ 007
240ml	305-9482	ELZ 008

**For further information about Irripod and Stericlens, please contact the UK Distributor:**M&A PHARMACHEM LTD, Westhoughton, Bolton BL5 2AL Tel: 01942 816184 Email: [info@mapharmachem.co.uk](mailto:info@mapharmachem.co.uk)

Irripod and Stericlens are a registered trade mark of CD Medical Ltd.